
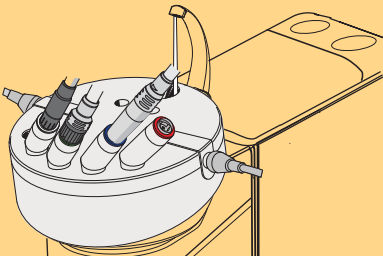

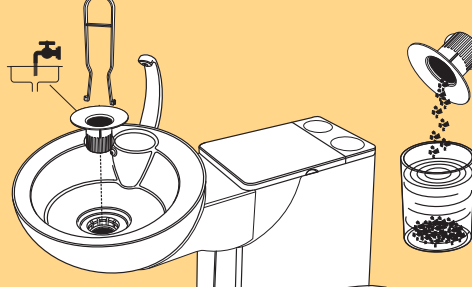
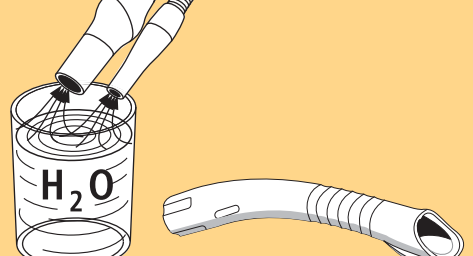
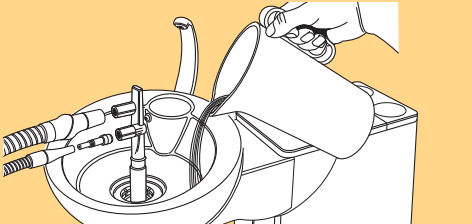
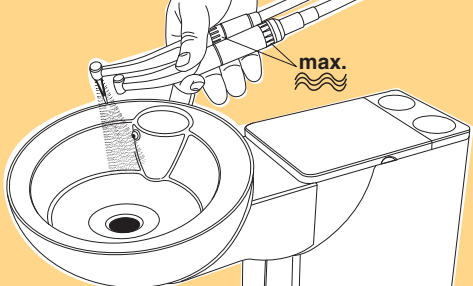
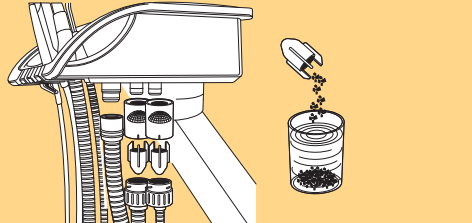

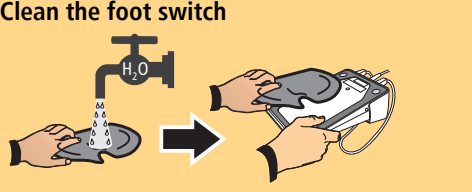
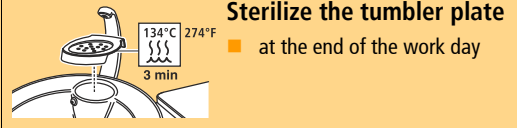
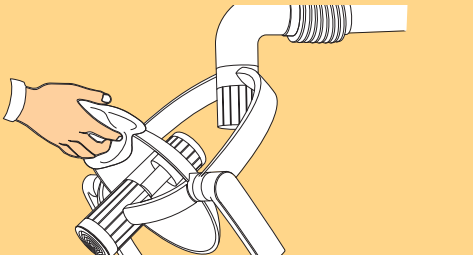
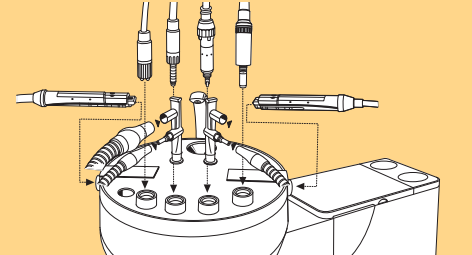
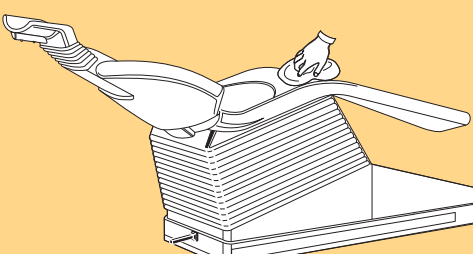
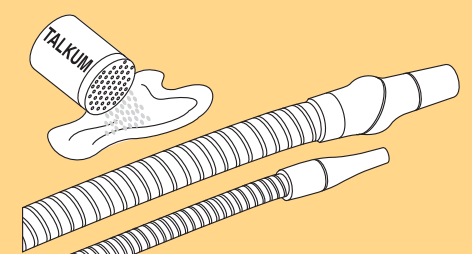
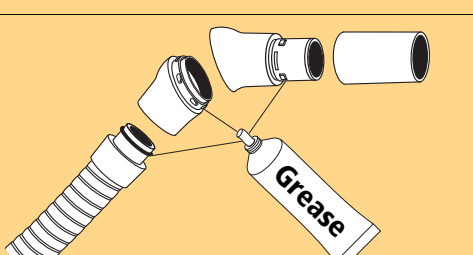
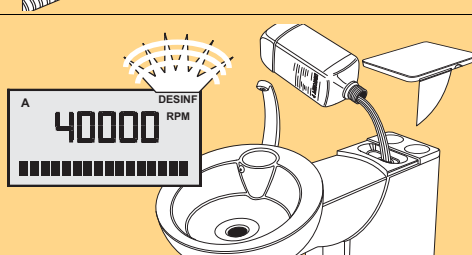
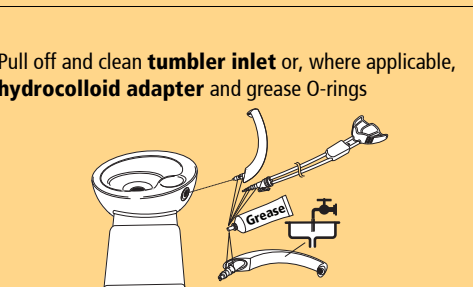
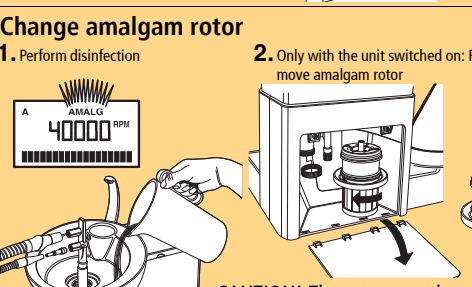


Care and Cleaning Plan for C2+ – C5+ and M1+ Treatment centers

A continuously updated list of approved care, cleaning and disinfecting agents can be downloaded from the Sirona internet website under REF 59 70 905: www.sirona.com

After each patient		Daily	
	<p>Disinfect and clean surfaces and upholstery</p> <p>⚠ ATTENTION Use only cleaning and disinfecting agents approved by Sirona!</p> <p>Headrest covers (5 pcs) for motorized headrest: C2+ – C5+ and M1+ disposable: Order No. 5872820 C2+ – C5+ and M1+ washable: Order No. 5905588</p>		<p>Auto Purge function (serially included in treatment centers with dentist unit as of Serial No. 75000)</p> <ul style="list-style-type: none"> before beginning the work day or after lengthy periods of disuse at the end of the work day <p>Automatic purging of all water carrying instruments of the dentist element, the assistant's Sprayvit and the tumbler filling unit</p>
	<p>Clean the cuspidor</p> <p>⚠ ATTENTION Use only cleaning and disinfecting agents approved by Sirona!</p> <p>Do not use any household or abrasive cleaners!</p>		<p>Remove the gold trap and clean it</p> <ul style="list-style-type: none"> during the noon break at the end of the work day <p>Collect amalgam residues in a closed container with water.</p> <p>⚠ ATTENTION CAUTION! Do not empty amalgam residues into the cuspidor.</p> <p>Pour 1 liter of water into the cuspidor to rinse it.</p>
	<p>Aspirate a large glass of cold water particularly:</p> <ul style="list-style-type: none"> after interventions during which the patient loses blood after using H₂O₂ and at least every 60 min. during prolonged treatments <p>To prevent reflux, please use the suction cannula with supplementary air intakes!</p>		<p>Aspirate a disinfecting agent</p> <ul style="list-style-type: none"> during the noon break at the end of the work day <p>⚠ ATTENTION Use only cleaning and disinfecting agents approved by Sirona!</p>
	<p>Purge function</p> <p>Rinse the water paths of the dentist element instruments that were used.</p>		<p>Clean the filter</p> <ul style="list-style-type: none"> at the end of the work day <p>Collect amalgam residues in a closed container with water.</p> <p>⚠ ATTENTION Do not empty amalgam residues into the cuspidor.</p>
			<p>Clean the foot switch</p>
			<p>Sterilize the tumbler plate</p> <ul style="list-style-type: none"> at the end of the work day <p>⚠ ATTENTION Do not operate the unit without the tumbler plate!</p>

Weekly		Monthly or as required	
	<p>Clean the reflector of the operation light</p> <p>⚠ ATTENTION Clean only with 70% alcohol!</p> <p>CAUTION! The reflector must be cold. For details on care and cleaning of the lamp, see the SIROLUX FANTASTIC Operating Instructions.</p>		<p>Sanitation</p> <p>For reduction of the biofilm within the fresh water carrying systems of the treatment center</p>
	<p>Care for and clean upholstery</p> <p>⚠ ATTENTION Only use care and cleaning agents approved by Sirona for upholstery! The sponge used with agent FD360 may not be used on lounge upholstery.</p> <p>Please follow the operating instructions of the cleaning and disinfecting agents precisely.</p>		<p>Suction hoses</p> <p>If the surfaces of the hoses have become sticky from disinfectants, clean them with a commercially available dishwashing liquid and then powder them with a light coat of talcum, if necessary.</p> <p>C3+, C4+ and C5+ with magnetic identification only: Do not treat these suction hoses (hard PVC) with talcum.</p>
	<p>Separate suction handpieces at joint; clean and grease points of separation</p> <p>⚠ ATTENTION Use only agents approved by Sirona!</p>		<p>Refill water disinfectant</p> <p>⚠ ATTENTION Use only agents approved by Sirona!</p> <p>DENTOSEPT P, 6x 1 liter: REF 33 18 156 in Canada/Japan/USA: DENTOSEPT P, 6x 1 liter: REF 59 69 535</p>
	<p>Cuspidor</p> <p>Tumbler filling unit with automatic sensor control: Clean area surrounding water outlets</p>		<p>Change amalgam rotor</p> <ol style="list-style-type: none"> Perform disinfection Only with the unit switched on: Remove amalgam rotor Fill in collected amalgam residues, close amalgam rotor. Remove amalgam rotor from lower part and send it for disposal. <p>CAUTION! The rotor must be replaced after 12 months in any case!</p> <p>Disposal</p>

For details, please refer to the corresponding documents "Care and cleaning instructions for the practice team".
The national requirements and recommendations for hygiene and disinfection must be observed (e.g. RKI, ADA, CDC, etc).