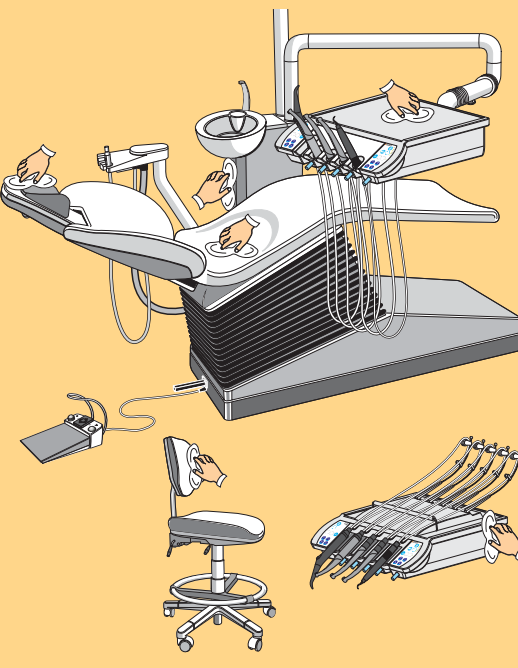
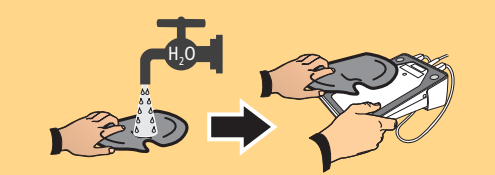
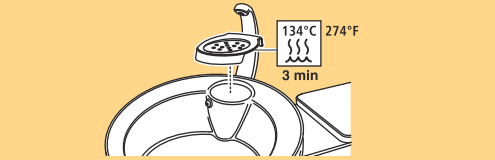
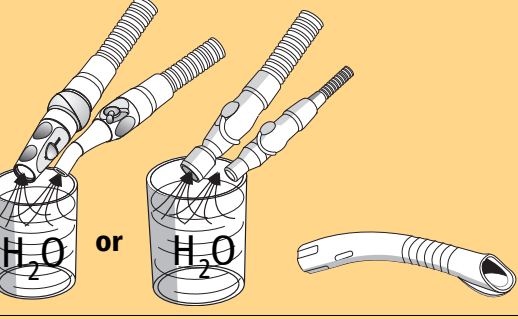
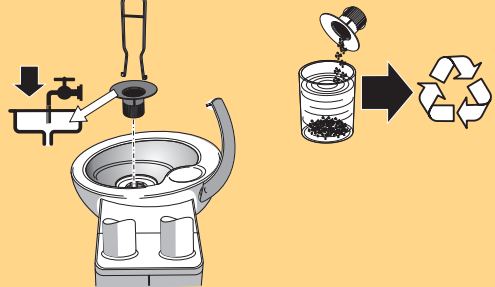
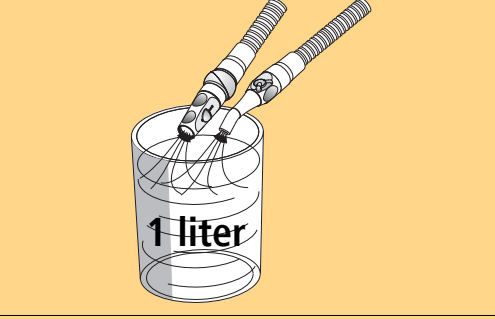
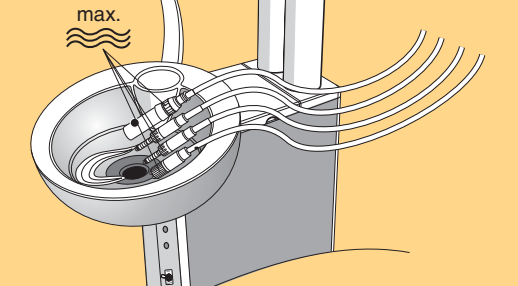
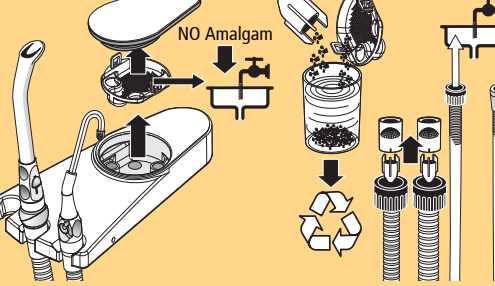
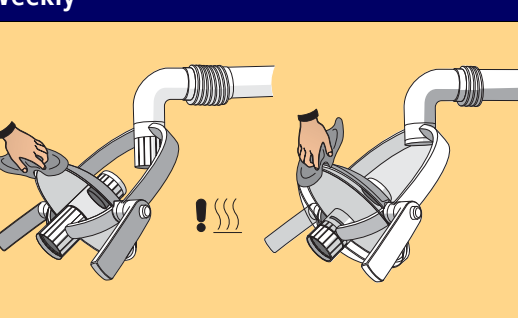
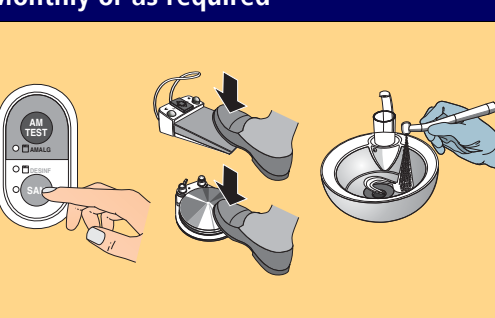
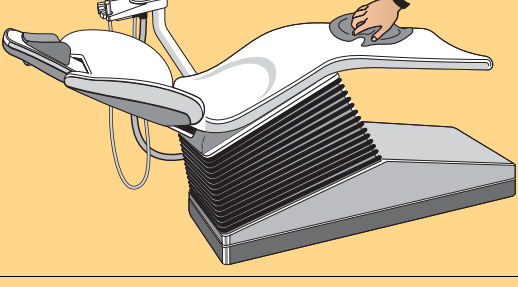
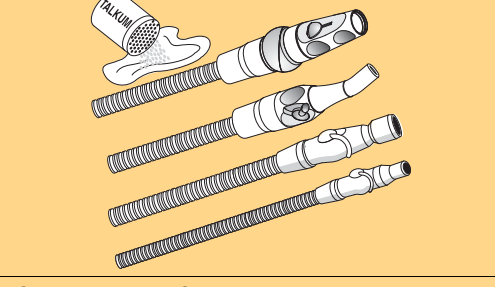
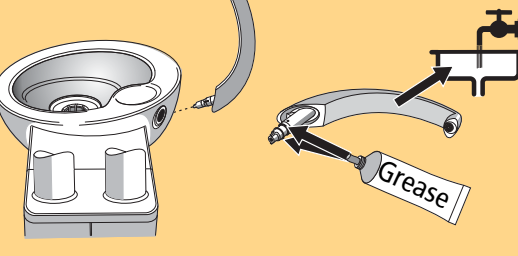
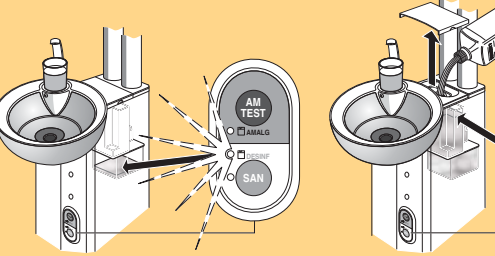
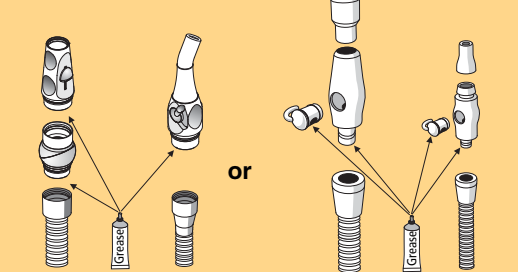



Care and Cleaning Plan for C8+ - Treatment centers

A continuously updated list of approved care, cleaning and disinfecting agents can be downloaded from the Sirona internet website under REF 59 70 905: ⇒ www.sirona.com

After each patient		Daily	
	<p>Disinfect and clean surfaces and upholstery</p> <p>⚠ ATTENTION Use only cleaning and disinfecting agents approved by Sirona!</p>		<p>Clean the foot switch</p>
	<p>Clean the cuspidor</p> <p>⚠ ATTENTION Use only cleaning and disinfecting agents approved by Sirona!</p> <p>Do not use any household or abrasive cleaners!</p>		<p>Sterilize the tumbler plate</p> <ul style="list-style-type: none"> at the end of the work day <p>⚠ ATTENTION Do not operate the unit without the tumbler plate!</p>
	<p>Aspirate a large glass of cold water particularly:</p> <ul style="list-style-type: none"> after interventions during which the patient loses blood after using H₂O₂ and at least every 60 min. during prolonged treatments <p>To prevent reflux, please use the suction canula with supplementary air intakes.</p>		<p>Remove the gold trap and clean it</p> <p>Collect amalgam residues in a closed container with water. CAUTION! Do not empty amalgam residues into the cuspidor. Pour 1 liter of water into the cuspidor to rinse it.</p>
	<p>Aspirate a large glass of cold water particularly:</p> <ul style="list-style-type: none"> after interventions during which the patient loses blood after using H₂O₂ and at least every 60 min. during prolonged treatments <p>To prevent reflux, please use the suction canula with supplementary air intakes.</p>		<p>Aspirate a disinfected agent</p> <p>⚠ ATTENTION Use only cleaning and disinfecting agents approved by Sirona!</p>
	<p>Purge function</p> <p>Rinse the water paths of the dentist element instruments that were used.</p>		<p>Clean the filter</p> <p>Collect amalgam residues in a closed container with water.</p> <p>⚠ ATTENTION Do not empty amalgam residues into the cuspidor.</p>
Weekly		Monthly or as required	
	<p>Clean the reflector of the operation light</p> <p>⚠ ATTENTION Clean only with 70% alcohol!</p> <p>CAUTION! The reflector must be cold.</p>		<p>Sanitation</p> <p>For reduction of the biofilm within the fresh water carrying systems of the treatment center</p>
	<p>Care for and clean upholstery</p> <p>⚠ ATTENTION Only use care and cleaning agents approved by Sirona for upholstery! The sponge used with agent FD360 may not be used on lounge upholstery.</p> <p>Please follow the operating instructions of the cleaning and disinfecting agents precisely.</p>		<p>Suction hoses</p> <p>If the surfaces of the hoses have become sticky from disinfectants, clean them with a commercially available dishwashing liquid and then powder them with a light coat of talcum, if necessary.</p>
	<p>Pull off and clean tumbler inlet or, where applicable, hydrocolloid adapter and grease O-rings</p>		<p>Refill water disinfectant!</p> <p>DENTOSEPT P, 6x1 liter: REF 33 18 156 in Canada/Japan/USA: DENTOSEPT P, 6x1 liter: REF 59 69 535</p>
	<p>Separate suction handpieces at joint; clean and grease points of separation</p> <p>⚠ ATTENTION Use only agents approved by Sirona!</p>		<p>Change amalgam rotor</p> <p>at least 1x annually</p>

For details, please refer to the corresponding documents "Care and cleaning instructions for the practice team".
The national requirements and recommendations for hygiene and disinfection must be observed (e.g. RKI, ADA, CDC, etc).