## Kurzanleitung PerioScan

### Bedienung:

#### Spülmedierversorgung:

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### Operation:

#### Rinsing medium supply:

Preselect the rinsing medium supply by pressing one of the **left keys**. The selected tank is illuminated from below.

#### Power setting in %:


Preselect the abrasive power in % by pressing one of the **keys in the bottom row**. The preselected power value is indicated digitally on the display. You can increase the power by:

- pressing and holding the preselected value or
- pressing and holding a higher value.

The power can be decreased by pressing and holding a lower value.

#### Detection function:

Press key **"1"** **shortly** to switch on and off.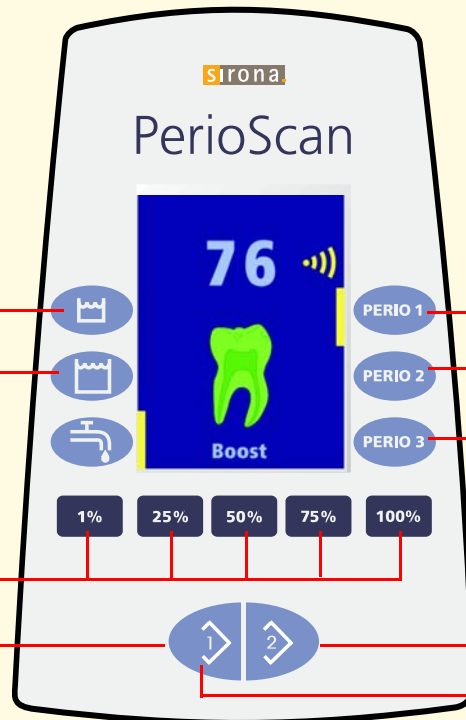

#### Reprogramming of program keys:

Select e. g. the rinsing media supply type and power value in %. Press the desired **PERIO key** for more than 3 sec. until the indicator bar next to the key flashes briefly. The setting is saved.

#### Continual rinsing:

Press key **"2"** for 60 s continual rinsing

*Can be terminated by pressing the foot switch or by pressing the key again.*

#### Learning program for probing pressure:

Press key **"1"** **long** to switch on and off.


*Can be terminated by pressing the foot switch.*

### Foot switch:

#### Boost function:

i.e. the preset power value is increased by 20: Actuate the **left pedal**.

**Normal operation**, i.e. abrasive power and cooling/rinsing media: **both pedals** must be pressed **simultaneously**.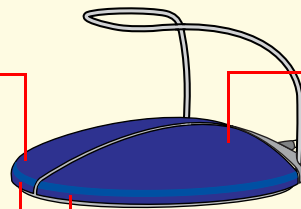

#### Rinsing:

when pressing the **right pedal**, only rinsing medium is discharged from the tip, flushing loose particles out of the treatment area. The vibration of the instrument is deactivated in this case.

### Key combinations:

#### Signal mode:

To switch the signal tone on or off, press **"Small rinsing media tank"** and **"PERIO 1"** **simultaneously**.

The signal tone can be assigned to the detection of **"calculus"** with the **"PERIO 1"** key or to **"noncontact of the tip"** with the root surface with the **"PERIO 2"** key.

It is possible to adjust the volume of the signal with: **"50%"** (quieter) and **"100%"** (louder).

#### Patient communication:

Press the **"Large rinsing media tank"** and **"PERIO 2"** keys **simultaneously** to switch the function on or off.

To switch between images, press the **"PERIO 1"**, **"PERIO 2"** and **"PERIO 3"** keys.

#### Demonstration mode for sales reps:


To demonstrate the detection function with replacement materials. Press **"Fresh water"** and **"PERIO 3"** simultaneously to switch on. *Press the foot switch to switch off.*

#### Boost function:


Press keys **"1"** and **"2"** **simultaneously** to switch function on or off.

#### Reset:

Switch the unit off. Then press and hold key **"1"** while switching the unit on.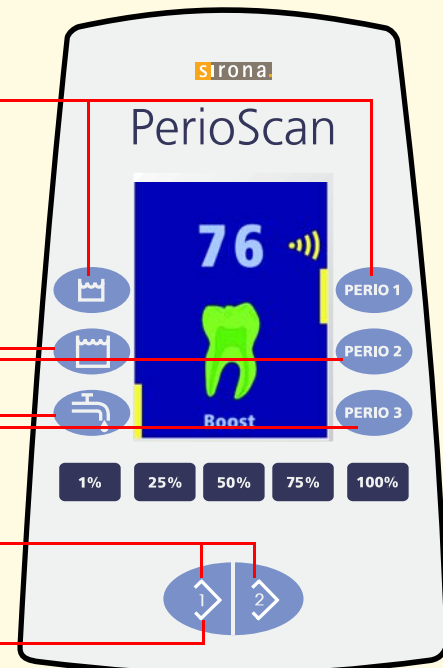


## ■ Use of the sealing ring

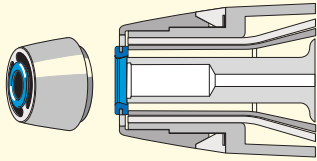
Before using the PerioSonic always be sure to verify the correct seating of the sealing ring and check its condition. If it is defective, damaged or worn, this can lead to false detection results.

## ■ Use of the 4PS detection tip

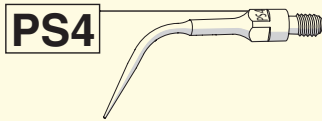
Detection is possible only with the **PerioSonic handpiece** and with a **SIROPERIO 4 PS tip** inserted. This tip can be recognized by the engraving shown.

## ■ Clinical use of the tips

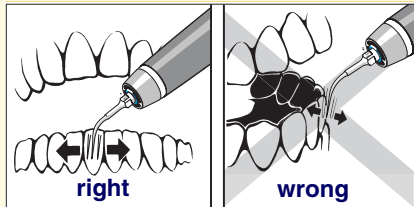The use of a piezoelectronically driven ultrasound scaler results in a linear vibration pattern. It is therefore important to apply the tip **parallel** to the tooth being treated and **never from the front**.

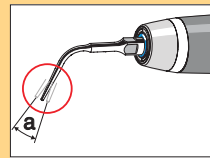**Please note that:** The detection function and the removal of calculus work only in the area where the tip vibrates.

## ■ List of approved rinsing media

- Hydrogen peroxide < 3 %
- Chlorhexidine < 0,3 %
- NaCl solution < 0,9 %
- Water

Hydrogen peroxide and chlorhexidine are mutually not compatible, i.e. you must flush with demineralized water for 60s before filling with the other rinsing medium ("continuous flushing" function). Please also flush for at least 10s even when changing over between compatible rinsing media.

## Cleaning and care of the unit

### Disinfecting the surfaces:

The surfaces can be disinfected with surface disinfectants (by spray and/or wipe). Do not pull on hoses or cables while cleaning.

**CAUTION:** Use only disinfecting, cleaning and care agents which are approved by Sirona! (see operating instructions).

### Cleaning the surfaces:

Remove any dirt and disinfectant residues regularly using a mild commercial cleaning agent. Do not use any colored cloths, since they can cause discoloration. Make sure that no liquids can penetrate into the interior of the unit.

### Cleaning the foot switch:

Clean or disinfect the bottom of the base plate using a water moistened cloth. Do not pull on the connection cable when cleaning it.

### Cleaning and disinfecting the rinsing media tanks:

The rinsing media tanks can be cleaned from the inside with hot water and a commercial dish washing liquid and a bottle brush. They are dishwasher proof and also can be thermally disinfected in a thermo disinfector at 95°C (203°F) and 10 min. holding time.

**CAUTION:** Perform thermal disinfection of the tanks only with removed covers.

## Cleaning and care of the handpiece

### Cleaning the surfaces:

Brush off the hand-piece under running water (of at least drinking water quality).

### Disinfecting the surfaces:

Steam and disinfect the hand-piece at 105°C with the tip and the sealing ring removed. Spray and wipe disinfection with chemical disinfectants is permitted only externally! Do not use any strong acid, alkaline solutions (5 < pH < 9) or solutions containing chlorine.

**CAUTION:** The hand-piece, tips and sealing rings must not be thermally disinfected! Never immerse them in disinfectants! Never clean them in an ultrasonic bath!

### Sterilizing:

The hand-piece, tips, sealing rings and torque wrench are sterilizable at temperatures of up to 135°C. Before sterilizing, unscrew the tip and blow any residual water out of the hand-piece and the tip.

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