Scaling Spitzen / Scaling tips

Spitze Nr. 1L / Tip no. 1L	REF 89 24 250
Spitze Nr. 2L / Tip no. 2L	REF 89 24 268
Spitze Nr. 3L / Tip no. 3L	REF 18 94 455
Spitze Nr. 4L / Tip no. 4L	REF 18 94 463

Endodontie Spitzen / Endo tips

Endospitze Nr. 5 / Endo tip no. 5	REF 60 02 310
-----------------------------------	---------------

SiroPerio Spitzen / SiroPerio tips

SiroPerio PE 1	REF 60 44 825
SiroPerio PE 2	REF 60 44 833
SiroPerio PE 3	REF 60 44 841
SiroPerio 4 PS	REF 61 77 005
SiroPerio PE 5	REF 63 43 078
SiroPerio PE 6	REF 63 43 086

SiroRetro Spitzen / SiroRetro tips

SiroRetro KL	REF 54 57 887
SiroRetro KR	REF 54 57 895
SiroRetro GL	REF 54 57 903
SiroRetro GR	REF 54 57 911

CEM-Set L / CEM L set

6L	REF 18 94 307
----	---------------

SiroPrep Spitzen / SiroPrep tips

SiroPrep M1	REF 59 04 276
SiroPrep M2	REF 59 47 358
SiroPrep D1	REF 59 47 309
SiroPrep D2	REF 59 47 366

Implantologie Spitze / Implantology tip

SiroImplant IP 1	REF 63 43 052
------------------	---------------

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Read the operating instructions for the ultrasonic handpiece carefully prior to first use of the tips and keep them handy for further reference.

Requirements for the safe use of Dentsply Sirona tips

See the operating instructions for care and maintenance tips. Observe all safety information and warnings, including the following:

Hygiene measures

- The ultrasonic handpieces and the dismantled tips should be sterilized after each patient to prevent any cross-contamination or transmission of infective agents such as HIV (the AIDS virus) or hepatitis B that might cause serious illnesses or even lead to death.
- New ultrasonic handpieces and the accessories used on the patient must be sterilized before use.
- When shipping the tip, make sure that it has been cleaned and disinfected.
- Flush all water lines for 2 minutes at the beginning of every workday.

Avoiding personal injury and material damage

- When the handpiece is not in use, remove the instrument tip or attach the torque wrench (REF 60 06 196) to prevent injury.
- The ultrasonic vibrations may damage the surface of the patient's teeth if used improperly.
- We recommend using a cofferdam when performing endodontic/implantology work. If this is not possible, the patient should breathe through his or her nose.
- Do not continue working in the case of malfunctions, irregular operating noise, or if the tip is damaged (bent).
- Tighten tips with the torque wrench and check them prior to each use by applying axial force.
- Make sure the instrument tip is in good condition and fits tightly prior to every use.
- Ensure that the CEM tip is screwed on tightly and does not come loose during treatment.
- Ensure that the tip is cooled sufficiently. If necessary, insert the cleaning wire into the water channel of the tip and blow air through it.

Working conditions

Ambient temperature:	10 °C – 40 °C (50 °F – 104 °F)
Relative humidity:	30% – 95%
Air pressure	700 hPa – 1060 hPa

Power setting for different tips

With just a few exceptions, a maximum power of 100% is possible for all the tips listed above. The exceptions are:

Tip no. 3L	up to 80%
Tip no. 4L	up to 70%
PE1	up to 70%
4PS	up to 80%
Endo tip no. 5	maximum 4% (SiroSonic/L and PerioSonic)
	SiroSonic L (Intego/Intego Pro) with endo function 1e to 5e.
	SiroSonic TL (Teneo/Sinius) with endo function 1e to 5e.
	NOTE: These values are not identical with the values 1 to 5 in ultrasonic operation.
CEM tip	up to 10% (SiroSonic/L/TL) up to 30% (PerioSonic)
	NOTE: The tip may come loose if the power is too high. This can result in the tip overheating.
IP 1	up to 70% (SiroSonic/L/TL) up to 20% (SiroSon L)

Special features of various tips

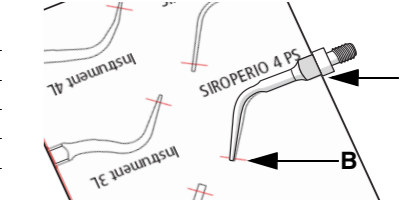
- The water flow is blocked when the CEM tip is used. Only brief, intermittent operation is allowed. Operate handpiece for only a few seconds at a time.
- Screw the endo tip no. 5 in and out only with the endo key (REF 89 28 418) and only at the square. Never screw it in or out at the head.
- SiroImplant IP 1 can be used only on the SiroSon L/SiroSonic/L/TL and PerioSonic. Always operate it with the max. coolant quantity. Prior to each treatment, please ensure that no metal is visible under the coating and tighten the holder for the insert with the knurled bushing and check it.

Checking tips

- Wear and bending of the tips lead to loss of performance and hazards.

Check regularly to ensure the tips are usable. Use the tip gauge (REF 54 55 667) for this. Place the tip on the tip gauge and align the edge (A) of the wrench surface (see illustration).

The marking lines (B) show the maximum permissible wear.

Any deviation from the contour shown indicates an unacceptable deformation. The illustrations on the tip gauge show the correct tip shapes and lengths.

- The tips may not be bent back into the proper shape. This would cause a risk of breakage during operation and result in no or lower detection performance (PerioScan).

To ensure successful treatment, make sure that the tip has sufficient diamond coating prior to use.

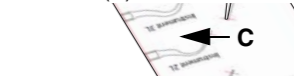
Service life of the tips / torque wrench

The tips and the torque wrench supplied with them reach their wear limit after 12 months of use and must be disposed of.

Exception: SiroImplant IP 1, after 1,000 treatments or 24 months.

Document and check the period of use of the tips and the torque wrench using the 4-digit alphanumeric serial number.

4-digit alphanumeric serial number (C) e.g. H6F1

Tip number (D) e.g. no. 3L

Disposal

The product must be disinfected prior to disposal.

Observe the applicable national regulations regarding disposal.

If you have any questions, contact your dental depot or the manufacturer.

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